

RAYON SWAB**Presentation**

Sterile Rayon swab with breakable plastic stick.

Sterilization method

Ethylene oxide.

Application

Used for collecting clinical samples from the oropharynx for microbiological tests.

Suitable for collecting oropharyngeal samples for the diagnosis of COVID-19 and other diseases.

Can be used for environmental monitoring to collect samples from surfaces.

Principle

Produced with non-toxic medical grade components, without autofluorescence. Suitable for sampling for qPCR-based assays.

How to use

Scrub the swab over the desired surface according to the adopted sampling plan. At the end of the collection, place the swab into adequate transport medium, break off the handle, and close the cap. Send the sample to the laboratory and proceed with the analysis according to the adopted methodology.

Precautions and special care

Product intended for *in vitro* diagnostic use only.

Restricted for use by professionals. Do not inhale or ingest.

Do not use the product beyond the expiration date, with signs of contamination, or if it has changed color. In the presence of contamination, the product should be immediately discarded.

Do not use the product if the packaging is damaged or tampered with.

Storage

Store at room temperature in a dry place and protect from light.

Disposal of the product

After use, the product must be handled at the generating unit before environmentally appropriate final disposal, in accordance with official regulations.

Quality Guarantee

bioBoaVista guarantees the quality of its products as long as they are used according to their respective instructions and in accordance with national and international references. bioBoaVista does not take responsibility for the use of its products for purposes other than those described and approved by the company. All clinical diagnoses should be analyzed in conjunction with clinical evidence and not solely based on laboratory results.